



TRANSPORT CHAIR INSTRUCTION SHEET

For Service or Questions (24 Hours):

888-573-1400

Purpose of Equipment: The Drive Medical Transport chair is to transport the patient for short distances. Example – bed to bathroom, car to doctors office, etc. This is not to be used as a regular wheelchair and self-propelling could cause tipping, so is **not** recommended.

OPERATING INSTRUCTIONS: The frame is made from aluminum, which gives the chair the lightweight feel, yet it is very sturdy and durable. Detachable footrests are set out further than most transport chairs, preventing the rider's heels from getting caught on the front wheels of the chair.

Light weight and extremely portable, the transport chair is able to fit into a trunk or backseat.

CLEANING INSTRUCTIONS:

Wipe down chair weekly or as needed using a damp cloth and mild detergent.

SAFETY PRECAUTIONS:

- Before entering or leaving the chair, engage the dual wheel locks against the tires on both rear wheels.
- Make sure the front casters are in the forward position before transferring into or out of the chair.
- Always fasten the safety belt when the transport chair is in motion.
- **Leaning out of the transport chair without proper assistance could cause tipping.**
- **Do not step** on the footplates when transferring, as this could cause tipping.
- **NOTE:** You and your attendant should consult a qualified professional for correct techniques on ramp negotiations.

Galaxy Medical distributes this product. Any questions or concerns can be addressed by calling our office, toll-free at **888-573-1400**