



BRODA CHAIR BASIC INSTRUCTION SHEET

For Service or Questions (24 Hours):

888-573-1400

Purpose of Equipment: Reduce patient slumping and sliding without the use of restraints. Designed to ease the burden on caregivers. Full recline allow for easy bed transfers and changing of incontinence products. Reduces Pressure. Creates Air Flow. Maximizes Comfort.

Features:

Infinitely adjustable tilt and recline; Broda Comfort Tension Seating; 6" Directional Locking rear casters; Removable/swing away side panels; Height adjustable arms; Durable 16 gauge steel frame; Dual flip down footrest; Angle adjustable footplates; Elevating & length adjustable leg rest; 20" seat Width; 17" seat depth; 20.5" seat height; 32" back length; Full, Adjustable Padding Including Upper and Lower Side Pads, Calf and Sole Pad, Terry cloth covered Seat and Back Pad.

SET UP INSTRUCTIONS:

Chair comes fully assembled; Our technician will help you adjust the chair to fit the patient.

CARE OF THE BRODA CHAIR:

Remove outer cover. Wash terry cloth cover in wash machine cold water. Line dry. Wipe down chair as needed with diluted disinfectant or mild soap.

ACCESSORIES:

Tray table; Padded thigh belt; Self releasing seat belt; Padded waist belt

This product is distributed by Galaxy Medical. Any questions or concerns can be addressed by calling our office, toll-free at **888-573-1400**