



48" Drive Med Aire Alternating Pressure System **For Service or Questions (24 Hours)** **888-573-1400**

Purpose of Equipment:

The Drive Med Aire is an alternating pressure relieving mattress replacement system used in the treatment and prevention of pressure sores. This product is not a substitute for a turning schedule and procedures for turning should be adhered to at all times. The mattress is designed to react to patient movement and automatically readjust air pressure.

Description: 10" tall air cells, 48" wide, 2" convoluted foam base, weight limit of 750 lbs.

Pump Instructions: Suspend the power control unit at the foot end of the bed or place on smooth flat surface. Plug into properly grounded electrical outlet. Connect the male/female couplers on the hoses from the mattress to the couplers on the bottom of the control unit.

ON/OFF Switch: Conveniently located on the right side of pump.

CPR Release: To rapidly deflate mattress, pull this.

Cleaning: Clean mattress cover weekly or as needed using a damp cloth and mild detergent. You may wash cover sheet with mild detergent and line dry. Wipe down motor as needed with a damp cloth. Remove filter from back of pump monthly by popping off filter cap. Can be hand washed gently with mild soap & water. Dry thoroughly

Power Failure/Transport: Will hold air for up to one hour. Cell in Cell design prevents patient from bottoming out in event of power loss.

SAFETY PRECAUTIONS:

The system has been designed to comply with regulatory safety standards including: UL544

Keep the pump away from sources of liquids. Do not expose the system, especially the mattress to exposed flames, such as cigarettes, etc. Do not store in direct sunlight. Do not use hyper carbonate and phenolic based cleaning solutions. Keep sharp object away from the mattress. Ensure that the system is clean and dry prior to storage. Store the pumps and mattress in the protective bag supplied. We recommend the use of side rails on the bed while the system is in use. Local policies regarding the use of the side rails should be taken into account.

Safety Warning: The cover of this product is vapor permeable but not air permeable and may present a suffocation risk. It is the responsibility of the care giver to ensure that the user can use this product safely.

Warnings: Electrical equipment may be hazardous if misused. The pump's case (back) should only be removed by authorized technical personnel. **DO NOT USE THE DRIVE PUMP IN THE PRESENCE OF FLAMMABLE GASES AND ANESTHETIC AGENTS.** **Caution:** Federal law restricts this device to sale by or on the order of a licensed practitioner. The drag handles are provided ONLY for transporting the mattress and patient across flat surfaces in an emergency and should NOT be used for any other purpose.

This product is distributed by Galaxy Medical. Any questions or concerns can be addressed by calling our office, toll-free at **888-573-1400**