



PORTABLE/BACK-UP OXYGEN INSTRUCTION SHEET

For Service or Questions (24 Hours):

888-573-1400

Purpose of Equipment: To provide a supply of oxygen to the patient when the concentrator is not accessible (i.e. traveling, transporting, power outage, etc.).

OPERATING INSTRUCTIONS:

PORTABLE/BACK-UP OXYGEN SYSTEM:

1. Place E-tank into E-Cart. Tighten bolt on the front of the E-cart to ensure the tank is secure.
2. Break tab off stem of tank
3. Unscrew regulator knob counter clockwise and place regulator overtop stem of tank. Make sure when performing this function that the prongs on regulator are facing the holes on the E-tank.
4. Once regulator is secure in holes of tank, tighten regulator knob by turning clockwise until secure.
5. Place cannula on end of regulator.
6. Take E-wrench and place on top of E-tank and turn the wrench to the right to allow oxygen supply to regulator.
7. Set liter flow by turning the knob on the regulator to the prescribed liter flow.
8. When finished using portable system, use E-wrench and place on top of E-tank, and shut-off oxygen supply by turning wrench to the left. Then turn regulator dial to the "OFF" position.
9. Store E-tank in a rack or lie flat in a secure area.

SAFETY PRECAUTIONS:

1. Do not leave a nasal cannula under bed coverings or chair cushions. If the unit is turned on but not in use, the oxygen will make the material flammable. Set the power switch to the "Off" position when unit is not in use.
2. This tank delivers high purity oxygen, which promotes rapid burning. Do NOT allow smoking or open flames within 25 feet of this tank or any oxygen carrying accessory. Use no oil, grease, or petroleum-based products on or near the tank.
3. Always store in a secure rack or lie flat in a secure area.
4. When finished using tank, always turn off oxygen supply to the tank.

This product is distributed by Galaxy Medical. Any questions or concerns can be addressed by calling our office, toll-free at **888-573-1400**