



AEROSOL MACHINE INSTRUCTION SHEET

For Service or Questions (24 Hours):

888-573-1400

Purpose of Equipment: This device is used in conjunction with a pneumatic nebulizer to produce a fine aerosol mist of medication for respiratory therapy, for both children and adults suffering from respiratory disorders such as asthma, allergies, etc.

OPERATING INSTRUCTIONS:

1. Place unit on firm horizontal surface so that controls are within easy reach.
2. Make sure the power switch is in the OFF position. Then plug power cord into an 115V, 60 Hz outlet.
3. Start the compressor by turning the switch to the ON position.
4. Start the compressor for 1.5 minutes to clear out any condensation inside the compressor.
5. Stop the compressor by turning the switch to the OFF position.
6. Connect one end of plastic tubing to compressor air outlet and the other end to Nebulizer. The Nebulizer should be one that is compatible with the technical performance specifications of the Aeromist LT unit.
7. Place prescribed medication in Nebulizer taking care not to accidentally spill any onto appliance. Secure T-piece to top of Nebulizer. Slide Mouthpiece into either end of T-piece. If using an Aerosol Mask, attach mask directly to Nebulizer, T-piece and Mouthpiece are not used.
8. Start the compressor again by turning the switch to the ON position.
9. Place Nebulizer mouthpiece between teeth and close mouth. If using an Aerosol Mask, place mask over mouth and nose.
10. As aerosol starts flowing, inhale deeply and slowly through mouthpiece or through mask.
11. When treatment is over, stop the compressor by turning the switch to the OFF position and remove the plug from the power supply. Disconnect Nebulizer from tubing. Clean Nebulizer and rinse Nebulizer jar after each use.

SAFETY PRECAUTIONS:

1. Do not use or leave unit around water.
2. This product should never be left unattended when plugged in.
3. Close supervision is necessary when this product is used by, on or near children or incapable persons.
4. Use this product only for its intended use as directed in this instruction sheet. Do not use attachments not recommended by the manufacturer.
5. Never operate this product if it has been dropped or damaged, or dropped into water.
6. Do not leave packaging elements (plastic bags, cardboard boxes, etc.) within the reach of children or incapable persons as they represent a potential source of danger.
7. Keep the appliance and the cord away from heated surfaces.
8. Never block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair and the like.
9. Never use while sleeping or drowsy.
10. Never drop or insert any object into any openings or hose.
11. Do not use outdoors or operate where aerosol (spray) products are being used. Air the room before use.
12. The appliance is not suitable for use in the presence of an inflammable anaesthetic mixture with air, oxygen or nitrogen protoxide.

This product is distributed by Galaxy Medical. Any questions or concerns can be addressed by calling our office, toll-free at **888-573-1400**