



SUCTION MACHINE INSTRUCTION SHEET

For Service or Questions (24 Hours):

888-573-1400

OPERATING INSTRUCTIONS:

****Before using suction unit, you must be under direct supervision of a trained clinician or trained by a health care professional.**

1. Connect either end of the connecting tubing to the top of the collection bottle connector then connect the other end to the bacteria filter.
2. The patient tubing should be connected to collection bottle at the outlet labeled (patient). Please assure that all connections are secure and without leaks before using.
3. Make sure the power switch located on the side of the unit is in the "OFF" position before connecting the unit to a power source.
4. Switch the power to the "I" position (on), the unit starts running.
5. Adjust the vacuum level from 0 to 520 mm Hg by turning the vacuum adjust knob (clockwise to increase vacuum and counter-clockwise to decrease vacuum). Referring to the vacuum gauge while setting the desired level of vacuum. To accurately read the gauge, block the patient end of the hose or cap off the collection bottle and allow gauge to reach a stable vacuum reading.

SAFETY PRECAUTIONS:

1. Do not use or leave unit around water.
2. Close supervision is necessary when this product is used by, on, or near children or invalids.
3. Use this product only for its intended use.
4. Never operate this product if: 1. It has a damaged power cord or plug, 2. It is not working properly, 3. It has been dropped or damaged, 4. It has been dropped into water.
5. Keep the power cord away from heated sources.
6. Never use while drowsy or asleep.
7. All suctioning should be done in strict accordance with appropriate procedures that have been established by a licensed medical authority.
8. Some attachments or accessories may not fit the tubing supplied. All attachments or accessories should be checked prior to use to assure proper fit.
9. Proper operation of this product is necessary to avoid physical harm. Improper use can cause personal injury and/or tissue damage. This product should not be used unless the operator has been instructed by a physician or other qualified health care professional.

This product is distributed by Galaxy Medical. Any questions or concerns can be addressed by calling our office, toll-free at **888-573-1400**