



TRAPEZE BAR INSTRUCTION SHEET

For Service or Questions (24 Hours):

888-573-1400

Purpose of Equipment: The Trapeze Bar is an important room accessory designed to help patients change positions while in bed, and aid in the transfer from bed to chair with minimum attendant assistance.

INSTRUCTIONS FOR USE:

After the Trapeze Bar is installed by Galaxy Medical Technician to the patient's desired position, use the triangle to hold on to, pull yourself up or turn for comfort.

NOTE: The trapeze bar is **NOT** a support, it is an aid to assist in patient transfer.

CLEANING INSTRUCTIONS:

If desired, wipe dust or any substance on the bar, off with a damp cloth.

SAFETY

- * **NEVER** attempt to adjust bar, contact Galaxy Medical for assistance.
- * If trapeze bar becomes loose, **DO NOT USE**, contact Galaxy Medical immediately.

This product is distributed by Galaxy Medical.
Any questions or concerns can be addressed by calling our office,
toll-free at **888-573-1400**