



# Liberty Y-Connector Instructions

1. Ensure you have all the items necessary for this procedure:
  - a) Liberty Double Lumen Tubing
  - b) Liberty Y-connector
  - c) 2 - Drains

(Figure A)

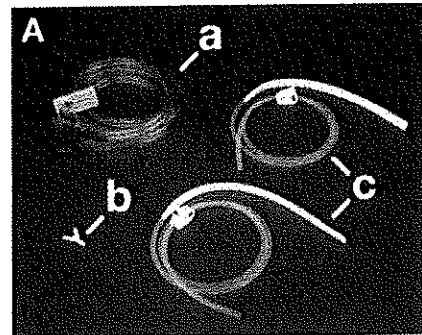


2. Remove one of the drains from the packaging. Using scissors, cut approximately 3 inches of tubing from the end of the tubing (Figure B).

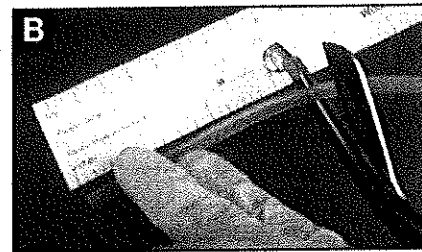

3. Attach the 3 inch piece of drain tubing to the bottom of the Y-connector (Figure C).



4. Attach the remaining two drains to the other ends of the Y-connector (Figure D).

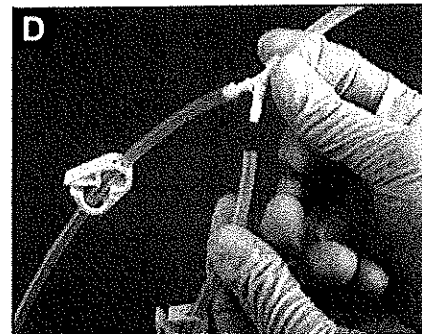

5. Attach the 3 inch drain tubing to the conical connector of the Double Lumen Tubing (Figure E).

